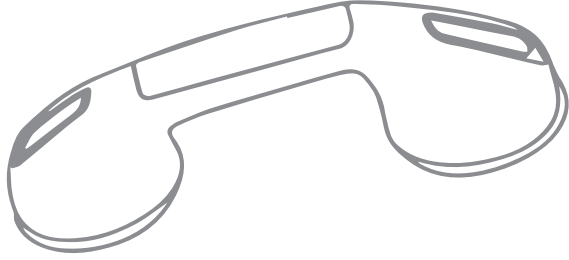
# Aprilla

ASH 9910  
Vantuzlu Kulp  
Suction Handle

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TR EN

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**Children Being Supervised not to Play with the Appliance**

*This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliances by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.*

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Hello,

First of all thank you for choosing our product. Please read this manual carefully before using your device. Please keep the manual as it may need in future. This guide is designed to help you to achieve maximum performance.

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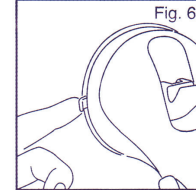
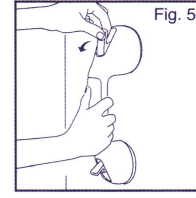
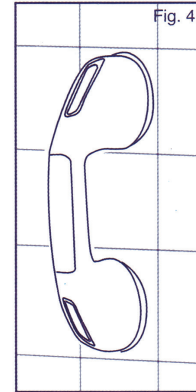
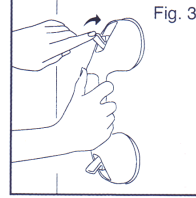
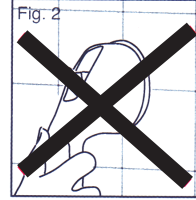
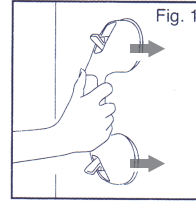
**WARNING**

This product is for temporary fixing and complementary support of the user. Do not use it as a "holding place" or "railing" in people with disabilities or elderly people. It is not designed to support lifting.

**CORRECT USAGE****ONLY USE AS INDICATED**

- Do not use to maintain leverage or balance
- Do not use in extreme temperatures.
- It should not be used by children.
- Do not use it to balance.
- Do not use under water.

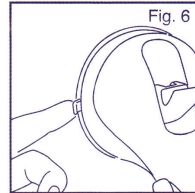
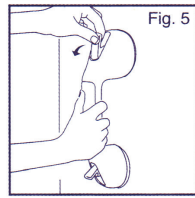
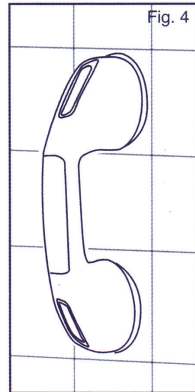
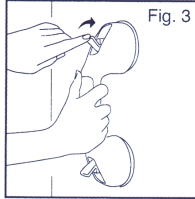
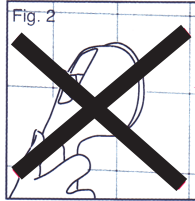
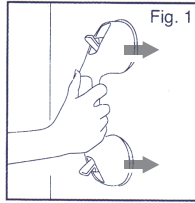
**CORRECT USAGE**

- Do not use on porous surfaces such as marble, granite, cement, cracked surfaces, painted surfaces, rough surfaces and wood.
- Do not use on weakened surfaces or loose tiles.
- Test the worm wheel before each use, and the product should be periodically restored in terms of hardness.
- Not suitable for outdoor use.

NOTE: Correct installation is necessary to ensure proper operation. Follow the installation instructions completely. If you encounter any difficulties during installation, use the services of a qualified builder.

**INSTALLATION**

- Make sure you are not in danger of losing balance when installing the vacuum housing.
- Find a clean, smooth surface for mounting your vacuum cup. Vacuum handle can be applied on flat tile and glass. Make sure that the surface to be trimmed is strong enough to support the use of the handle.
- Clean the surface of any dust, dirt or soap residue and let it dry completely.
- Raise the two latch arms to loosen the vants and place them on a smooth surface. (Fig. 1) Vantuz should be placed on a completely smooth surface. Do not stick to cracks or lines between tiles. (Fig. 2) Incorrect positioning of the vacuum handler results in malfunction and / or injury.
- Press the vacuum handle surface while closing both latch arms. (Figure 3)
- The installation shown in Figure 4 is shown. (Figure 4)

**ATTENTION**

MANUAL TEST BEFORE USING ANY USE, SURE THAT YOU WILL BE SURE AND USE FROM END USE. Re-install if you have no problems.

**FEATURES**

- Fast fixing on smooth, clean and smooth surfaces. Ideal for safe use of shower and toilet.
- By locking the latches, the fixing in the selected place is ensured. Easily remove the latches open.
- No tooling, holes or screws required.
- Ideal for use in bathrooms, showers, toilets and baths.
- Able to for transporting glass and smooth plates.

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**Correct Disposal of this product**

This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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© 2017 BMVA. Tüm hakları saklıdır. Aprilla logosu BMVA'ya aittir. Diğer tüm logo ve markalar sahiplerine aittir. Yazım ve baskı hatalarından BMVA sorumlu değildir. © 2017 BMVA. All rights reserved. Aprilla logo is owned by BMVA and registered. All other trademarks and signs are property of their owners. BMVA is not responsible for printing and composing mistakes.